

One Step Pregnancy Test Strip

Please read this leaflet carefully before performing this test.

INTENDED USE

The One Step Pregnancy Test Strip detects levels of the pregnancy hormone Human Chorionic Gonadotrophin (hCG) in your urine at a level of 15mIU/ml. HCG is a hormone that is produced soon after conception and progressively increases during the first three months of pregnancy. Very reliable and simple to use, each test comes individually sealed and is clearly labelled HCG STRIP ONE STEP URINE PREGNANCY TEST

PROCEDURE**MATERIALS PROVIDED**

- 1 x test strip
- instructional leaflet

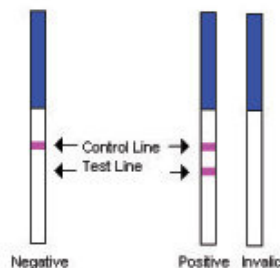
MATERIALS REQUIRED BUT NOT PROVIDED

- specimen container
- timer

DIRECTIONS FOR USE

1. A urine specimen of at least 5ml must be collected in a clean and dry container. A first morning urine specimen is preferred since it generally contains the highest concentration of hCG; however, urine specimens collected at any time of the day may be used.
2. Remove the test strip from the sealed pouch and use it as soon as possible.
3. With arrows pointing toward the urine specimen, immerse the test strip vertically in the urine specimen for at least 3 seconds. Do not pass the maximum line (MAX) on the test strip when immersing the strip. See the illustration below.
4. Place the test strip on a non-absorbent flat surface, start the timer and wait for the red line(s) to appear. The result should be read at 5 minutes. It is important that the

background is clear before the result is read. Do not read after 10 minutes.

**INTERPRETATION OF RESULTS**

POSITIVE: Two distinct red lines appear. One line should be in the control region (C) and another line should be in the test region (T).

NEGATIVE: One red line appears in the control region (C). No apparent red or pink line appears in the test region (T).

INVALID: Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test with a new test strip. If the problem persists, discontinue using the test kit immediately and contact your local distributor.

NOTE: The intensity of the red color in the test line region (T) will vary depending on the concentration of hCG present in the specimen. However, neither the quantitative value nor the rate of increase in hCG can be determined by this qualitative test.

FREQUENTLY ASKED QUESTIONS**When can I do the test?**

You can test as early as 7 days before your missed period. E.g. if your period is due on the 21st day of the month, you can test as early as the 15th day of the month. You can use One Step Pregnancy Test Strip any time of the day. You don't have to use first morning urine. Generally, however, your first morning urine contains the highest level of pregnancy hormone.

What do I do if the result is positive (pregnant)?

Remember, this test is not intended to replace your doctor's diagnosis. A positive (pregnant) test result shows that the pregnancy hormone (hCG) was detected. See your doctor to confirm that you are pregnant and for proper pregnancy care and nutrition counselling.

What do I do if the result is negative (not pregnant)?

You may not be pregnant, or it may be too early to tell. If you do not get your period within seven days, you should retest with another One Step Pregnancy Test Strip. It is possible that you miscalculated the length of your cycle or your urine may not have had enough pregnancy hormone for the One Step Pregnancy Test Strip to give a positive result. If you retest and again no hCG is found, and your period still has not started, you should call your doctor.

Will the result change if left standing for a certain period of time?

The test result should be read as soon as possible after waiting the five minutes. A positive result will last for at least 48 hours. If negative (i.e. 1 pink line) the result will last for 10 minutes. After 10 minute period, disregard any further changes, *it is still a negative result.*

Does the use of alcohol or medications affect the result?

Only medications containing the pregnancy hormone (hCG) can affect the result. Alcohol, pain killers, antibiotics or the contraceptive pill should not affect the result.

LOT NUMBER: 20070516

EXPIRY: 16 May 2009

Do not use after the expiry date printed on the bottom of the package.

Store in a dry place below 30°C. Do not freeze.

Keep out of reach of children.

For In-Vitro Diagnostic Use. Not to be taken internally.

Made in China by Runbio Biotech Co Ltd for ARG Group.

Distributed by: ARG Group

57 Salween Crescent Roxburgh Park

Victoria Australia 3064

AUST L 143328